

A-dec Monitor Mounts



A-dec monitor mounts use one of the following positioning mechanisms:

Floating

These mounts provide continuous tension which holds the monitor in place after it is positioned.





Locking

These mounts include a handle that allows you to manually lock the tilt position of the monitor.



A-dec monitor mounts are VESA (Video Electronics Standards Association) compliant, so they support most monitor brands and sizes.




NOTE A-dec monitor mounts are intended to support and position a medical grade or equivalent flat-panel monitor. For monitor requirements, please see the table on page 3.


Monitor Mount Adjustments and Specifications

For adjustment information and monitor specifications for your mount, see the images below and the table on page 3.


Style 1, Floating

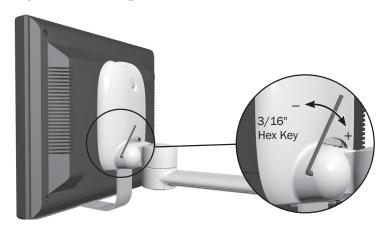


Style 2, Floating

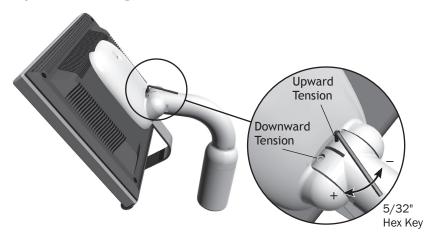

Style 3, Floating

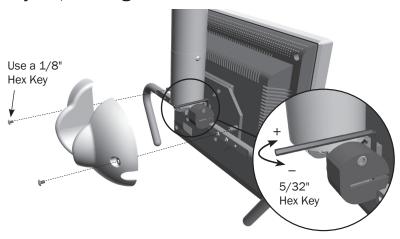

Style 4, Locking

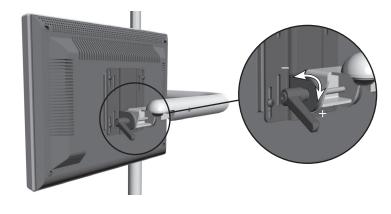

Vertical Tension Adjustment	Hex Key Size	Adjustment Style	Available On	Maximum Monitor Weight	Maximum Monitor Size
Style 1, Floating Turn the hex key clockwise to increase or counterclockwise to decrease the tension.	3/16"	Floating / Single-Tension	chairs, support centers	20 lb (9 kg)	19" (483 mm) diagonal
Style 2, Floating Turn the hex key clockwise to increase or counterclockwise to decrease the tension. This adjustment may be necessary for both the upward and downward tension.	5/32"	Floating / Dual-Tension	chairs, support centers, cabinet track-mount	31 lb (14 kg)	19" (483 mm) diagonal
Style 3, Floating Use a 1/8" hex key to remove the back cover screws. Use a 5/32" hex key to increase or decrease the mount's vertical tension.	1/8" 5/32"	Floating / Single-Tension	ceilings, cabinets, walls	31 lb (14 kg)	19" (483 mm) diagonal
(Note: The handles may be repositioned after the cover is removed. Use a 1/8" hex key to remove and replace the handle screws.)					
Style 4, Locking Turn the handle clockwise to lock the monitor position.	N/A	Locking/Handle	chairs, support centers	31 lb (14 kg)	19" (483 mm) diagonal

86.0309.00 Rev E

Specifications & Warranty



IMPORTANT For electrical specifications, identification of symbols, and other regulatory requirements, see the *Regulatory Information, Specifications, and Warranty* document (p/n 86.0221.00), which is available in the Document Library at www.a-dec.com.



NOTE Specifications are subject to change without notice. Requirements may vary depending on your location. For more information, contact your authorized A-dec dealer.

Warranty information is provided in the *Regulatory Information*, *Specifications, and Warranty* document (p/n 86.0221.00), which is available in the Document Library at www.a-dec.com.

Product Models and Versions Covered in This Document

Models	Versions	Description
300, 381, 382, 482, 581, 584, 585, 586, 587, Performer 8000	A	Monitor Mount

a dec

A-dec Headquarters

2601 Crestview Drive Newberg, Oregon 97132 United States

Tel: 1.800.547.1883 within USA/CAN Tel: +1.503.538.7478 outside USA/CAN

Fax: 1.503.538.0276 www.a-dec.com

A-dec Australia

Unit 8 5-9 Ricketty Street Mascot, NSW 2020 Australia

Tel: 1.800.225.010 within AUS
Tel: +61.(0).2.8332.4000 outside AUS

A-dec China

A-dec (Hangzhou) Dental Equipment Co., Ltd. 528 Shunfeng Road Qianjiang Economic Development Zone Hangzhou 311106 Zheijiang, China Tel: +1.503.538.7478

A-dec United Kingdom

EU Authorized Representative
Austin House, 11 Liberty Way
Nuneaton, Warwickshire CV11 6RZ
England
Tel: 0800.ADEC.UK (2332.85) within UK
Tel: +44.(0).24.7635.0901 outside UK





86.0309.00 Rev E Copyright 2016 A-dec Inc. All rights reserved.