



www.nsk-inc.com

700 Shimohinata Kanuma-shi Tochigi 322-8666,

Japan
TEL: +81 (0)289-64-3380
FAX: +81 (0)289-62-5636
FAX: +49 (0)6196 77606-0
FAX: +49 (0)6196 77606-29

www.nsk-europe.de

Elly-Beinhorn-Strasse 8

65760 Eschborn.

Germany

Office 5, Gateway1000, Arlington Business Park, Whittle Way

Stevenage, SG1 2FP, UK
TEL: +44 (0)1438 310670
FAX: +44 (0)1438 310671

NAKANISHI INC. | NSK Europe GmbH | NSK United Kingdom Ltd | NSK Dental LLC | NSK Oceania Pty Ltd | NSK Middle East | NSK Asia www.nsk-uk.com

1800 Global Parkway Hoffman Estates, IL 60192,

TEL: +1 888 675 1675 FAX: +1 800 838 9328

www.nskdental.com www.nskoceania.com.au www.nsk-inc.com Unit 22, 198-222 Young St. Waterloo, Sydney, NSW 2017, Australia

TEL: +61 (0)2 8306 3000 FAX: +61 (0)2 8306 3030

Room 6EA-701, 7th Floor,

East Wing No.6 Dubai Airport Free Zone, PO Box 54316 Dubai, UAE TEL: +971 (0)4-204-5507 FAX: +971 (0)4-204-5508

www.nsk-inc.com

1 Maritime Square, #09-33 HarbourFront Centre, Singapore 099253

TEL: +65-6376-5018 FAX: +65-6376-0233

Specifications are subject to change without notice.

REF PR-D772E Ver.2 14.00.00.00

iCare

iCare sets a new standard for handpiece maintenance systems. In one stylish and compact body, it offers you both ease of use and multiple functions to achieve complete cleaning and lubrication. The proper solution volume is preset based on handpiece type, and maintenance can be initiated at the touch of a button. The chuck cleaning system keeps chucks pristine, and the purging select mode lets you alter purge time depending on the situation for truly comprehensive maintenance. Thanks to its stylish and compact body, iCare is also space efficient and totally in harmony with your working environment.

ADVANTAGES

- Preset solution volume based on handpiece type (high-speed handpieces, low-speed handpieces)
- Thorough cleaning and lubrication with simple operation
- Chuck cleaning function ensures that the chuck is thoroughly cleaned and lubricated
- Purging select mode ensures complete removal of excess oil
- Integrated large oil tank (1.2 litre) reduces burden of refill



AUTOMATICALY CLEANS & LUBRICATES

PRESET PROPER SOLUTION VOLUME BASED ON HANDPIECE TYPE

The proper solution volume is preset based on high speed and low speed handpieces, ensuring precise and thorough lubrication at the touch of a button.




ERGONOMIC AND EASILY READABLE LARGE PANEL Simple One-Touch Operation to Start Lubrication.

LOW SPEED HANDPIECES



Low-speed handpieces incorporate gearing systems that require extra solution penetration. The NSK rotation system guarantees optimum oil delivery to the internal components of all low-speed handpieces.

HIGH SPEED HANDPIECES

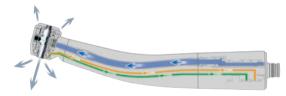

iCare meticulously cleans the components of high-speed handpieces.

SSURE ARE S

PURGING SELECT MODE

After lubrication, you can select from appropriate modes offering different purge lengths.



CHUCK CLEANING FUNCTION

The chuck cleaning function completely removes dirt and cutting powder from the chuck.



EASY TO CLEAN

iCare's front door can be completely removed to allow fast, easy cleaning of the system.



HP HEAD FILTER PREVENTS MIST LEAKS (Optional)

HP Head Fllter is made to absorb any spray mist generated, which prevents mist leaks.



SOLUTION TANK

iCare's big solution tank has a capacity of 1.2 liters, allowing you to pour in a full one-liter bottle of maintenance solution at once, reducing the necessity for frequent refilling. The enclosed nozzle enables clean and easy refills.

SAFE OPERATION WITH A RANGE OF ALARM FUNCTIONS

The alarm LED ensures safe and secure maintenance by alerting you to various situations, including inadequate solution quantity and an unlocked front door.

ACCOMMODATES BOTH NSK HANDPIECES AND THOSE OF OTHER BRANDS

NSK and all other brands of low speed handpieces connect directly to the coupling supplied. NSK high speed handpieces require the PTL adaptor. Adaptors are available for direct connection of most other brands of high speed handpieces.



AUTOMATICALY CLEANS & LUBRICATES



STYLISH AND COMPACT

In the pursuing ultimate ease of use, NSK has given iCare the power to perform automatic maintenance for up to four handpieces at once. iCare is also designed to be simple to set up and space efficient, you can install it virtually anywhere.



iCare COMPLETE SETS

Dual Standard Ports + Dual Rotation Ports

MODEL ORDER CODE iCare C2 Type (120V) Y1002796 iCare C2 Type (230V) Y1002787



CONTENTS

- iCare unit
- Power cord
- Mist filter set
- Oil absorber sheet
- Air tube
- Maintenance oil for iCare (1Litre) Chuck Cleaning Nozzle

Single Standard port + Triple Rotation ports

Y1002788

ORDER CODE iCare C3 Type (120V) Y1002797



SPECIFICATIONS

iCare C3 Type (230V)

Power Supply: AC 100-240V 50/60Hz

• Air Pressure: 0.5 - 0.6 MPa (5.0 - 6.0 kgf/cm²)

Weight: 6.5 kg (C2 type/C3 type)

• Dimension (mm): W280 x D240 x H350

• Solution Tank Capacity: 1,200 mL

* Connects only to Midwest 4-hole type

* Air piping is necessary (5.0 - 6.0 kgf/cm²)

iCare ADAPTORS

MODEL	ORDER CODE		
PTL Adaptor	Z257010	• For NSK High Speed Handpieces for PTL-CL-LED Coupling	
B2/M4 Adaptor	T904	• For Borden 2 hole High Speed Handpieces	
KV Adaptor	Z257020	 For KaVo® High Speed Handpieces for MULTIflex® LUX Coupling 	
SR Adaptor	Z257023	 For Sirona® High Speed Handpieces for Quick Coupling 	
WH Adaptor	Z257021	• For W&H® High Speed Handpieces for Roto Quick Coupling	
BA Adaptor	Z280022	• For Bien-Air® High Speed Handpieces for Unifix® Coupling	
F-Type Adaptor	Z280042	• For ENDO-MATE TC2, DT / TASKAL Wizard Heads	

Check with the dealer for the adaptors to mount other manufactures' handpieces.

SPARE PARTS

MODEL	ORDER CODE	
Maintenance Oil for iCare 1 Bottle, 1 Litre	Z016130	
Maintenance Oil for iCare Pack of 6 Bottles	Y1002783	(1Litre)
Mist Filter Set Pack of 12	Y900159	
Oil Absorber Sheet Pack of 10	U433135	
Air Filter	U508352	
Chuck Cleaning Nozzle	Z257301	
HP Head Filter Pack of 200	U1120118	

KaVo® and MULTIflex® LUX are registered trademarks of Kaltenbach & Voigt GmbH & Co. Sirona® is a registered trademark of Sirona Dental Systems GmbH. W&H® is a registered trademark of W&H Dentalwerk Bürmoos GmbH. Bien-Air® and Unifix® are registered trademarks of Bien-Air Dental S.A