


## Panorama exposure

### Preparation

Select support.

- 1 Bite piece
- 2 Chin cup
- 3 Chin rest

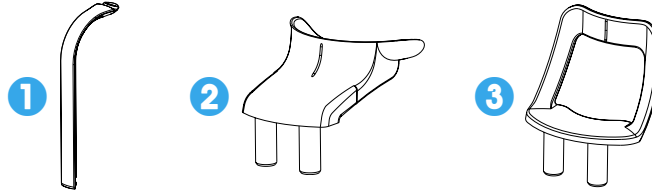

Select program.


Basic program (Standard)	<i>Traditional beam path and angles.</i>
Interproximal panoramic program (Advanced program, optional)	<i>Used in the detection of caries.</i>
Orthogonal panoramic program (Advanced program, optional)	<i>X-ray beam perpendicular to the jaw. Practical for examination of the gums.</i>
Panoramic program for Bitewing (Bitewing, optional)	<i>Produces images similar to Bitewing images.</i>

Select patient size.



### Patient positioning

- 1 Position the patient's head so that the Frankfort plane is aligned with the Frankfort plane light beam.
- 2 Position the patient's head so that the midsagittal plane is aligned with the midsagittal plane light beam.
- 3 Position the patient so that the apices of the upper central incisors are within the unit's image layer (focal trough).

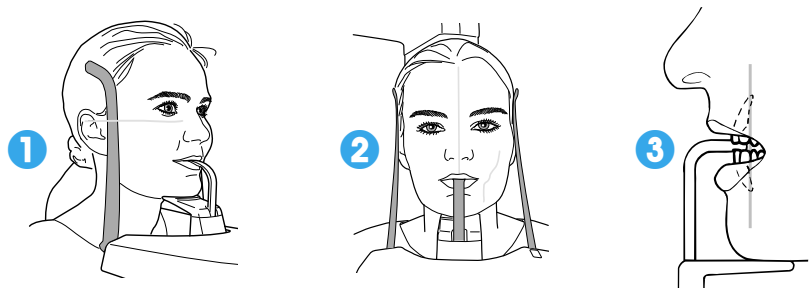

### Exposure

Press the exposure button and hold down for the duration of the exposure.

30003299\_1/9.2016/en