

ASK SIMON

THE DENTAL EXPERT



DENTAL CHAIR WATERLINE MAINTENANCE

The system has 3 processes:

1. Flush/blast the waterlines to remove all biofilm
2. Maintenance of lines to help slow down biofilm growth
3. Test – to make sure low instances of biofilm in the lines

RO-Water, using A-dec tablets and other products all help with the maintenance of the lines.

1. Testing Water

- Use test slides to test water for biofilm build up every quarter

2. If needed, flush/blast the waterlines

- For 1st chair buy Alpron & Biofilm removal starter kits - includes fluids & measuring jugs etc.
- For 2nd, 3rd etc. chair buy Clean & Clear re-treatment kits – just fluids if you have jugs from previous kit



3. Daily maintenance of lines to help slow down biofilm growth

- Option 1 – Most Popular - A-dec ICX Tablets – just add tablet to bottles
- Option 2 - 5 Litre Alpron Concentrate – mix concentrate with water

Don't Forget!
Check your oil collector every week.

If very oily, check:

- Handpieces have been stood upright before going into tubing
- Check coupling/air

Contact Dental Medical Ireland today
Tel: 1890 400 405

