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
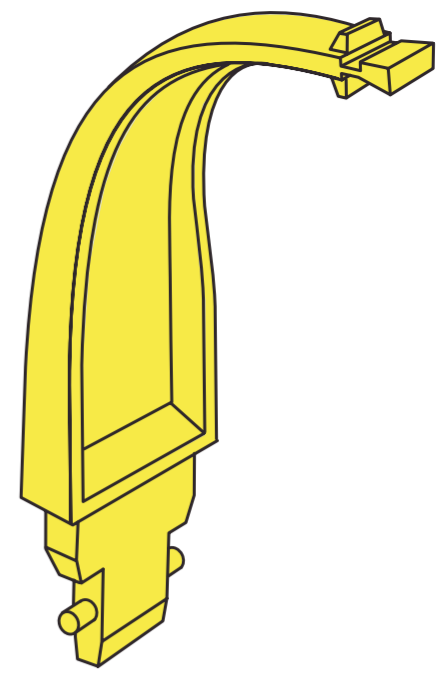
2D X-ray with Orthophos

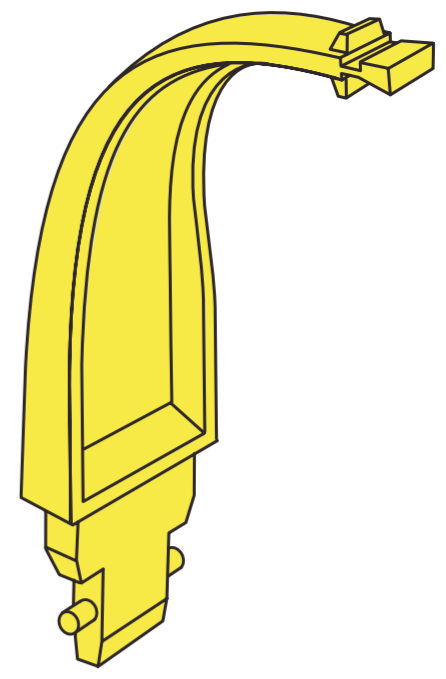Patient positioning with standard bite block for the perfect panoramic image


Preparation: Remove glasses, piercings, jewelry, and any removable dental prostheses, then put on an X-ray apron.

Hygiene protection: The appropriate hygiene protection is to be attached before each exposure. This must be disposed of immediately after use.


1 Front teeth of the upper and lower jaw

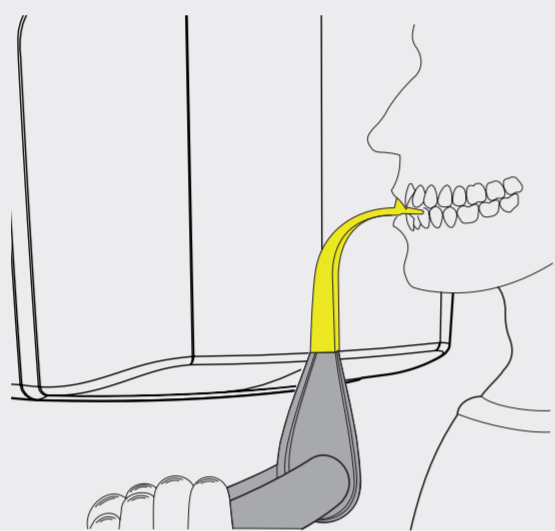Position the front teeth of the upper and lower jaw in the marked notch of the bite block. The front teeth are then positioned exactly in the sharp slice. Ask the patient to hold the hand grips firmly with both hands. Then swivel the control mirror outwards.


2 Correct standing position

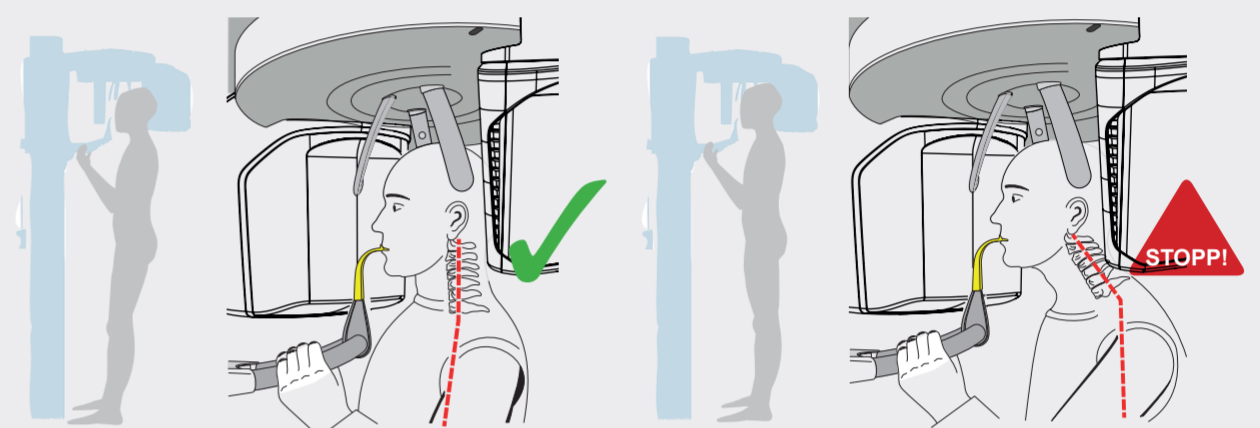Ask the patient to take a half step forward to extend the cervical spine. Extending the cervical vertebrae prevents lucencies on the front teeth in the X-ray image.


3 Checking the midsagittal plane

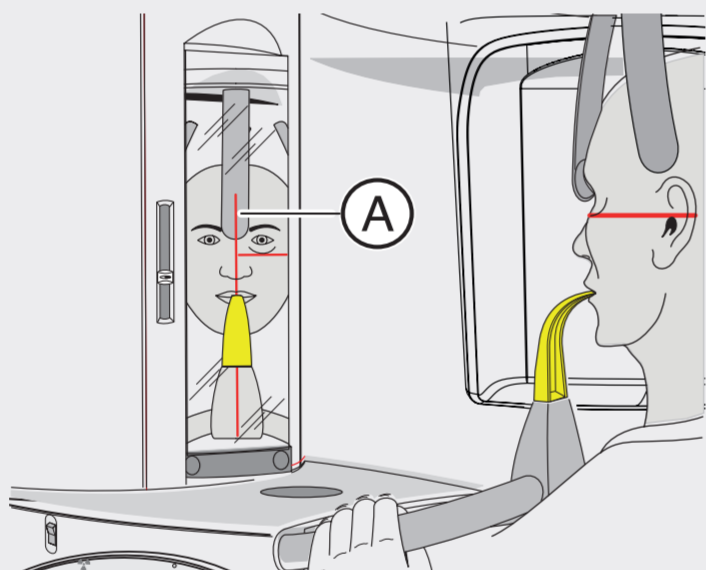The light localizer is used to ensure symmetry. Align the patient (nasal root) with central light line (A) (midsagittal line).


4 Checking the Frankfurt horizontal plane

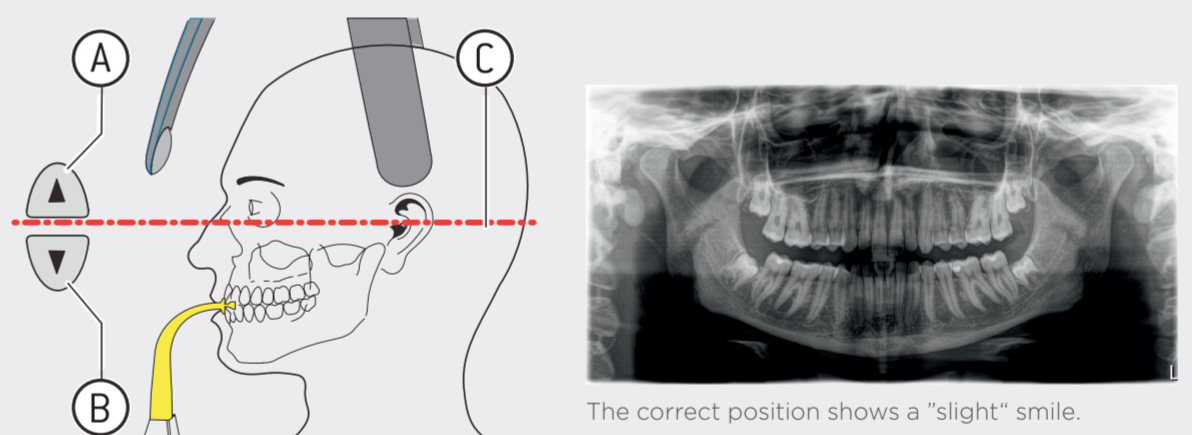The Frankfurt horizontal plane (C) runs from the left infraorbital margin to the upper margin of the external ear canal (porion). Correct the patient's head inclination as necessary. Press the height adjustment buttons if necessary, up (A) or down (B).


5 Head fixation

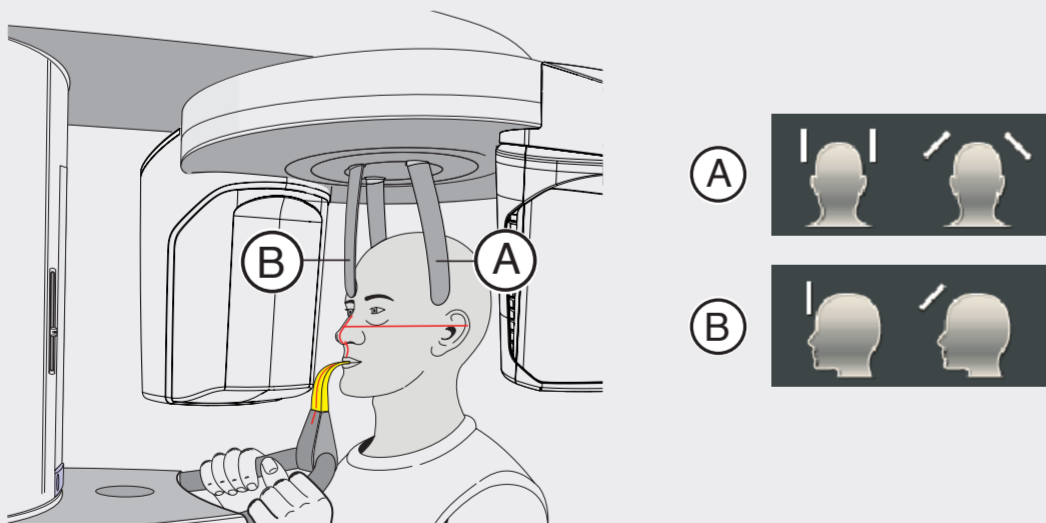Check the midsagittal plane again. First close the temple supports (A) and then move the forehead support (B) until it just contacts the patient. Swivel the mirror back.

6 Before taking X-ray

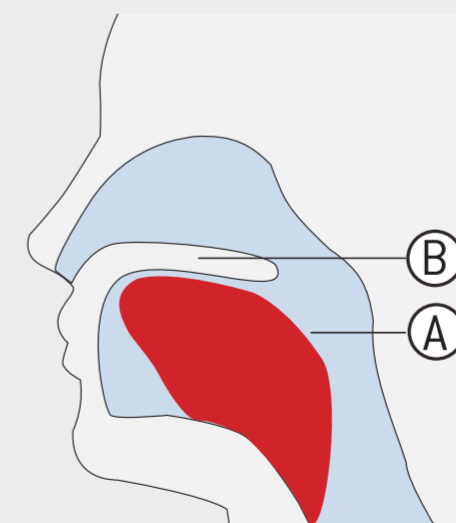Instruct the patient to place his tongue (A) against the roof of his mouth (B), and to hold this position until the end of the exposure.

Please read the instructions for use for a more detailed description.

Caution: Strong disinfectants can corrode the surface of the device. Therefore do not use substances containing phenol, peracetic acid, peroxide or other oxygen-splitting agents, sodium hypochlorite or iodine-splitting agents. Use only cleaning and disinfecting agents approved by Dentsply Sirona! You can find more information at dentsplysirona.com/careagents