

Disinfectant for Dental Vacuum and Amalgam Separators

New PULI-JET PLUS

1 litre makes up to 125 litres of disinfectant or 250 litres of cleaner



- 1 Unscrew red cap slightly.
- 2 Squeeze bottle to distribute liquid to top container.

- 3 To mix, fill the 'Pulse Cleaner Bucket' with warm water (**approx temp 50°C**) up to the third marker, then pour the Puli-Jet Plus into the 'Pulse Cleaner bucket'.

General Cleaning: We recommend that you cleanse the aspiration system **every evening**, and at the **end of work**. For this mix 10ml of Puli-Jet Plus with 2.5 litres of warm water.

Disinfecting: We recommend to disinfect the aspiration system **once a week** with a higher concentrate of Puli-Jet Plus. For this mix 20ml of Puli-Jet with 2.5 litres of warm water.

- Non-foaming concentrate
- Without aldehydes and phenol
- For disinfecting and simultaneous deodorization, cleaning and maintenance of dental aspirators and amalgam separators (wet, semi-wet & dry systems)
- Please always refer to the manufacturers instructions when using Puli-Jet Plus



LASTS
LONGER
THAN OTHER
COMPETITOR
BRANDS



Cattani ESAM (UK) Limited, 21A Harris Business Park, Hanbury Road, Stoke Prior, Bromsgrove, Worcestershire, B60 4DJ

Tel: 01527 877997

Fax: 01527 839799

Email: info@cattaniesam.co.uk

Web: www.cattaniesam.co.uk