Use Orotol® Plus daily and MD 555 weekly

After every treatment: aspirate 1 glass of water. Before every disinfection: aspirate 2 litres of water.

Daily use of Orotol® Plus (2%)

To be used after the end of treatment. Where there are higher levels of contamination, use twice daily.





Fill the OroCup with cold water up to the 2 litre mark.

Add 2 dosing caps of Orotol® Plus (only fill up to the mark).

- 3. Shake contents in the OroCup.
- 4. Aspirate the solution using the OroCup and allow it to work overnight.
- 5. Disinfect the spittoon with the remainder of the solution (1 litre in the OroCup).
- 6. Next morning aspirate 2 litres of water.









Weekly use of MD 555 special cleaner (5%)

To be used once a week. Where there are higher levels of contamination, use daily before the lunch break.

- 1. Aspirate 2 litres of water before cleaning.
- 2. Mix 2 litres of a 5% strength solution of MD 555
 Fill the OroCup with cold water up to the 2 litre mark.
 Add 5 dosing caps of MD 555 or 1 sample bottle
 (only fill up to the mark).
- 3. Shake contents in the OroCup.
- 4. Aspirate the solution using the OroCup and allow it to work over the lunch break.
- 5. Clean the spittoon with the remaining 1 litre of solution.
- 6. Aspirate 2 litres of water after the lunch break.







Notice to the installer:

Please ensure
this is given to the user
or left on display
with the pump.