



ORO CLEAN® Liquid



for daily use

ORO CLEAN® Plus




once a week

CLEANING

Highly effective concentrate for the cleaning of dental suction units, spittoon bowls, and amalgam separators.

- 1 Rinse the suction unit thoroughly with water after surgical procedures and prior to cleaning.
- 2 For cleaning, prepare 2 litres of working solution (to measure the correct dosage, use lid as a measuring cup).
- 3 Aspirate 1.5 litre of the working solution using the OROCLEAN® Cup and pour the remaining 500 ml down the spittoon bowl.
- 4 Leave to act for 30 minutes.
- 5 Rinse thoroughly with water after cleaning.

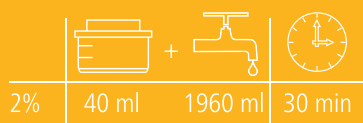

DISINFECTION & CLEANING

Highly effective concentrate for the disinfection and cleaning of dental suction units, spittoon bowls, and amalgam separators.

- 1 Rinse the suction unit thoroughly with water after surgical procedures and prior to disinfection.
- 2 For disinfection, prepare 2 litres of working solution (to measure the correct dosage, use lid as a measuring cup).
- 3 Aspirate 1.5 litre of the working solution using the OROCLEAN® Cup and pour the remaining 500 ml down the spittoon bowl.
- 4 Leave to act for 30 minutes.
- 5 Rinse thoroughly with water after cleaning.

